



- STERILIZATION CASE - Extra Small (706-XS, Boot & Clamp Only)
- Small (706-S, Base Only)
- Medium (706-M, Accessory Case)
- Large (706-L, Accessory Case)

Cleaning & Sterilization Procedure

NOTE: ALL SOLUTIONS MUST BE COMPATIBLE WITH ALUMINUM & STAINLESS STEEL.

Recommended Hand Cleaning Instructions.

- Soak the product in an enzyme solution. Follow the manufacturer's direction for dilution and soaking time.
- Put thru washer / sterilizer cycle, continue as normal procedure.
- Rinse the product in warm tap water.
- Soak or wipe the product down with a hospital approved and / or EPA approved germicide.
- Rinse the product in warm tap water.
- Wash the product with an instrument detergent up to a pH of 9.0
- RINSE THE PRODUCT IN WARM TAP WATER.
- DRY THOROUGHLY AND WRAP ACCORDINGLY:
48" x 48" WRAP. (706-XS, 706-S, 706-M) 54" x 54" WRAP (706-L)

Recommended Sterilization Instructions.

- Insure that all parts are thoroughly cleaned.
Make sure that all movable parts are loose and can move freely.
- DO NOT PLACE POSITIONER IN A MILK BATH OR LUBRICATE.
- Place Knee Positioner into Sterilization Case.

Double wrap in two disposable or reusable wraps.

Run normal vacuum cycle for your institution.

Minimum recommended parameters are as follows:

Pre-Vac Sterilization

| | |
|----------------|--------------------------------|
| EXPOSURE TIME: | 4 MINS |
| TEMPERATURE: | 270* - 272* F 133* - 134* C |
| DRYING TIME: | 20 MINUTES |
| COOLING: | @ ROOM TEMPERATURE |