




De Mayo Expand-A-Peg™ Locking Peg

INSTRUCTIONS FOR USE

1. Place foam or pressure reducing gel pad on top of pre-assembled pegboard.




2. Slide the *imp* Patient Protective Pad over the Peg with the perforation away from the patient. Separate the perforation enough to access the locking cap.

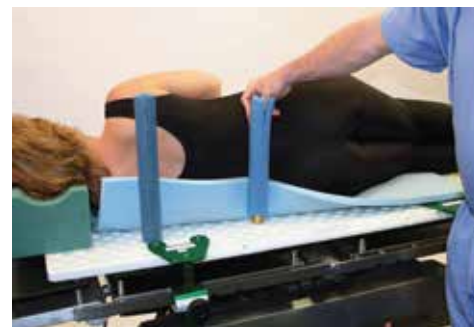

3. With the Expand-A-Peg™ Locking Peg in a "loose" position, insert the Peg at an angle locating the closest hole to the patient and securely seat the Peg.

4. Tighten the locking cap to lock into the peg hole.

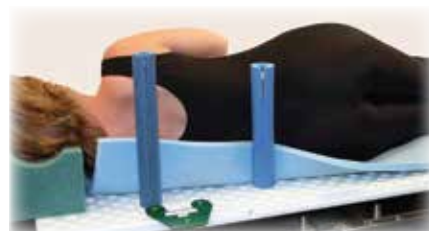

5. Anterior Pegs are typically placed in front of the pubis and superior to the pectoral muscle. Posterior Pegs are typically placed in the lumbar region and below the scapula. Additional pegs can be utilized for larger patients.

NOTE: WARNING. LIABILITY VOIDED IF *imp* PATIENT PROTECTIVE PADS ARE NOT USED.

De Mayo Expand-A-Peg™ Locking Peg is a trademark of Innovative Medical Products, Inc.
 PATENT APPLIED FOR
 MorphBoard® Positioning System is a registered trademark of Innovative Medical Products, Inc.
 MorphBoard® Positioner U.S. Patent No. 7,100,225
 © Innovative Medical Products, Inc. ALL RIGHTS RESERVED
 April 2018

imp Innovative
 Medical Products, Inc.

87 Spring Lane, PO Box 8028
 Plainville, CT 06062

PH: (860) 793-0391 FAX: (866) 459-1805

Toll Free PH: 1-800-467-4944 FAX: 1-888-229-1452
www.impmedical.com

