



# Instructions For Use

Humbles LapWrap®

Humbles LapWrap® Extensions

Catalog Number **904-G, 904-G EX**

Doc Title **IFU-IMP-0027**

Revision **01**



**Intended Use/Intended Users:** Humbles LapWrap is a medical device to be used to secure patients' arms by their sides during surgery. Orthopedic surgeons are the intended users.

**Target Patient Group:** Humbles Lapwrap is universally sized to accommodate any patient. LapWrap Extensions can be used for bariatric patients.

**Contraindications:** Humbles LapWrap should NOT be used to hold a patient in Trendelenburg or Reverse Trendelenburg positions.

**Warnings/Precautions**

- Always follow the Instructions for Use (IFU) for proper setup and correct product usage.
- Untrained personnel must review and understand the IFU
- Improper application may result in patient injury, including nerve damage, impaired circulation, or pressure injuries
- Do not overtighten; ensure adequate circulation is maintained
- The LapWrap is not a substitute for standard patient restraints where required
- Continuous patient monitoring is required during use
- Discontinue use immediately if signs of patient distress or compromised circulation occur
- Max number of reuses:
  - Single use

**Risks:**

- Hooks could break
- Fabric could discolor when exposed to light

**Complaints and Adverse Events:** For complaints and adverse events, contact IMP and the appropriate regulatory authorities for specific country.



Innovative Medical Products  
 87 Spring Lane  
 Plainville, CT 06062  
 P: 860-793-0391  
 F: 866-459-1805  
 info@impmedical.com

**Product Identification**

Part No	Product Name	GTIN
904-G	Humbles LapWrap®	00696588003559
904-G-EX	Humbles LapWrap® Extensions	00696588003566












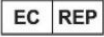

**Consumables:** Not Applicable

**Disposal of unit:**

Humbles LapWraps are to be disposed of in accordance with applicable laws, rules, and regulations for the disposal of biohazardous waste. Follow all guidelines for biohazardous waste in accordance with the Centers for Disease Control and Prevention guidelines as well as applicable federal/national, state and local regulations.







## Symbol Glossary




Symbol	Title	Description	Standard
	Authorized Representative in the European Community	Indicates the authorized representative in the European Community.	ISO 15223-1:2021
	Batch Number	Indicates the manufacturer's batch code so that the batch or lot can be identified.	ISO 15223-1:2021
	Catalog number	Indicates the manufacturer's catalog number so that the medical device can be identified. The manufacturer's catalog number shall be placed after or below the symbol and adjacent to it.	ISO 15223-1:2021
	Caution	Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot be presented on the medical device itself.	ISO 15223-1:2021
	Complies with European Directives.		
	Consult instructions for use		ISO 15223-1:2021
	Date of manufacture	The date must be presented in the following format: YYYY-MM-DD	FDA 21 CFR 801
	Keep Dry		ISO 15223-1:2021
	Manufacturer	This symbol shall be accompanied by the name and address of the manufacturer.	ISO 15223-1:2021
	Medical device		ISO 15223-1:2021
	Not made with natural rubber latex		Manufacturer defined
	Serial Number	The manufacturer's serial number shall be placed after or below the symbol and adjacent to it.	ISO 15223-1:2021
	Unique device identifier		ISO 15223-1:2021



**Instructions for Use:**

<p>1. Place LapWrap® on OR table pads, black hook fastener side UP.</p> 	<p>2. Place patient so the upper edge of the LapWrap® is located in the axillary region.</p> 
<p>3. Wrap entire 3 piece strap around the patient's arm and attach hook-and-loop fastener near or under patient.</p> 	<p>4. For unusual table positions, patient can be secured to the table by placing center tongue between OR table and side rail. Wrap tongue around side rail and attach black hook strap to green loop.</p> 

**For Bariatric Patients**

<p>1. Lift LapWrap® center tongue of the upper arm and pull between the torso and the biceps.</p>	
<p>2. Insert the tongue out through the cutout, while pulling the hook-and-loop fastener around the outer arm and towards the body.</p>	
<p>3. Fasten to the center tongue.</p>	



**Use of extensions are used to secure arms of the extremely obese patient**



1. After properly positioning patient on Lapwrap®, attach the extensions to the center tab.



2. Pull extension between the patient's arm and torso



3. Pull extension through the cut out of the LapWrap® wing.



4. Fasten the wing to the center tongue



5. If necessary, remove excess extension with standard scissors





**Optional for additional security**

6. Remove hook from extension and place hook on the opposite side of extension



7. Position extension between side rail and OR table.



8. Lock extension back down on green loop. Patient is now secured to the OR table.



**Optional Technique**


Place TUFFease® pressure reducing gel pad on OR table for extra patient protection.



**Safety Test**



Check that black hook is securely attached to green loop material at all locations.

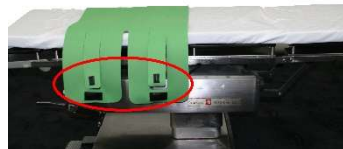

**Cleaning and Sterilization Procedure**

Not applicable – single use



**Scan for additional documentation**




**Innovative Medical Products**

87 Spring Lane  
Plainville, CT 06062  
USA

P: 860-793-0391

F: 866-459-1805

[info@IMPmedical.com](mailto:info@IMPmedical.com)

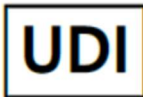

(01) 0 0696588 00355 9

(11) 260415

(10) M1234567



LapWrap® is a registered trademark of Innovative Medical Products, Inc.  
TUFFease® is a registered trademark of Innovative Medical Products, Inc.  
IMP® products are protected by patent & patent pending rights ~ go to [IMPpatents.com](http://IMPpatents.com)  
© April 2026 Innovative Medical Products, Inc.  
ALL RIGHTS RESERVED