




Positioned to Perform

Innovative Medical Products™

# CLOCK THE BOOT

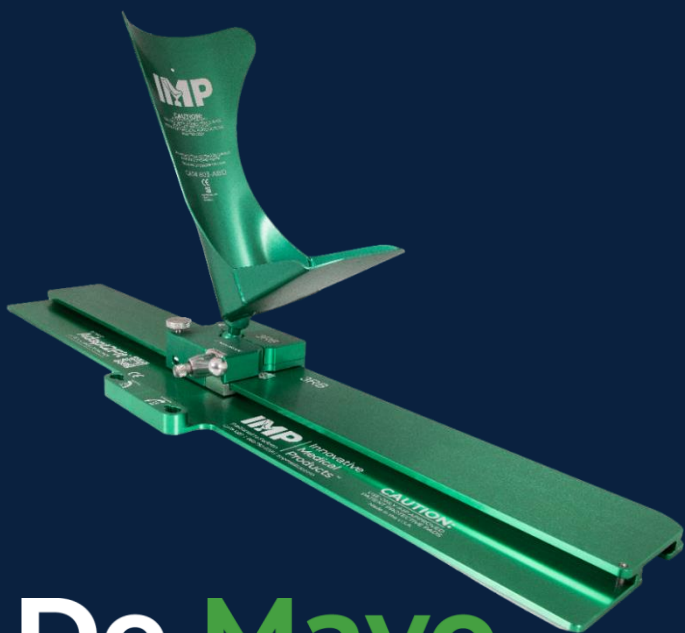

## De Mayo KNEE POSITIONER®

87 Spring Ln, Plainville, CT 06062  
(800)467.4944  
www.IMPmedical.com  
customerservice@IMPmedical.com

Catalog Number: 803-114, 1020  
Doc Title: INFO-IMP-0011  
Version: 0




Ensure optimal performance of the positioner in every procedure by clocking the boot

## STEPS

### 1 Place Carriage on Baseplate (Applicable to De Mayo D2)

A) From the “Loading End” (right side of the Baseplate) while holding the Teflon Pad underneath the carriage, B) slide the carriage around the curved end first until fully on the Baseplate.

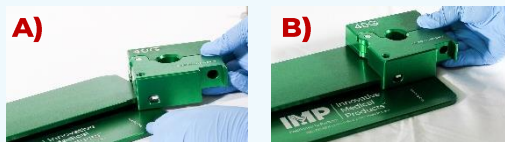


### Place Carriage on Baseplate (Applicable to Adapt2Fit)

While holding the Teflon Pad against the carriage, either: A) From the right end of the left Baseplate, slide the carriage on Baseplate or B) From the right end of the right Baseplate, slide the carriage around the curved end and onto Baseplate.



2 A) Using your fingers, screw the threaded rod with L-Plate into the carriage until the threads of the rod catch the inside of the carriage. B) Insert boot ball into carriage and continue to turn until finger tight. **Note:** If boot ball does not fit into carriage, loosen the threaded rod

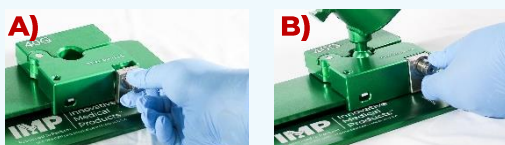

3 A) Place the handle in the 11 - 1 o'clock range and B) screw in the Thumbscrew with attached spring until finger tight. The handle now becomes a spring-loaded handle. C) Screw the E-Brake into the carriage.



## FUNCTIONAL TEST BEFORE USE

- 1 From the 11 - 1 o'clock position, turn carriage handle clockwise until handle stops. Boot is securely locked when the handle is in the 12 - 3 o'clock range.
- 2 Turn the handle counterclockwise, lift the boot out of the carriage, and slide the carriage freely on the base plate while the handle is unlocked.

