

CLOCK THE BOOT



De Mayo

KNEE POSITIONER®

87 Spring Ln, Plainville, CT 06062

1-800-467-4944

www.IMPmedical.com

customerservice@IMPmedical.com




CAT #: 803-114, 1020

Doc Title: INFO-IMP-0011



R01

STEPS Ensure optimal performance of the positioner in every procedure by Clocking the Boot.

- 1** **A)** Using your fingers, screw the threaded rod with L-Plate into the carriage until the threads of the rod catch the inside of the carriage. **B)** Insert boot ball into carriage and continue to turn until finger tight. **C)** Place the handle in the 9 o'clock range until they are mated. **D)** Screw thumbscrew with attached spring until finger tight. The handle now becomes a spring-loaded handle.

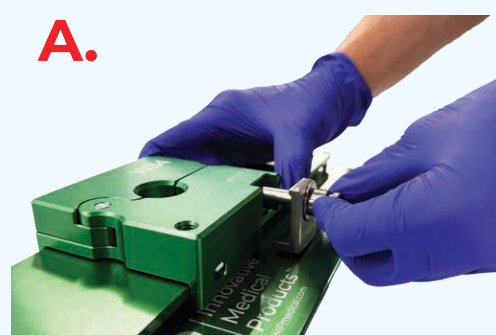

- 2** **A)** Grab base of handle and pull it towards yourself. **B)** Clock the handle between 2 and 3 o'clock. **C)** Screw the E-Brake into the carriage.

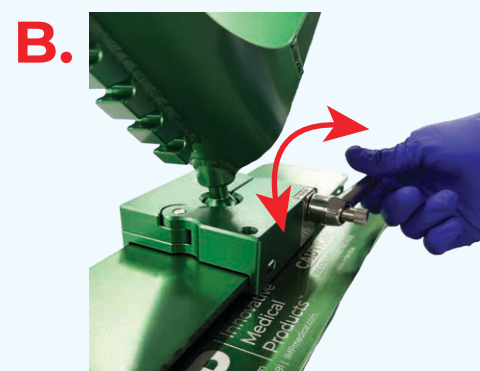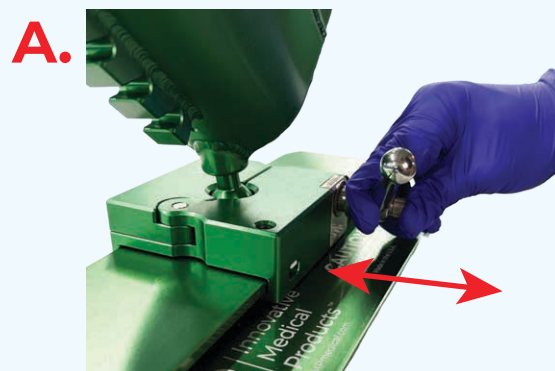

FUNCTIONAL TEST BEFORE USE

- 1** Boot securely locked when the handle is in the 2 - 3 o'clock range.
- 2** Turn the handle counterclockwise, lift the boot out of the carriage, and slide the carriage freely on the baseplate while the handle is unlocked.

