

# *imp*® De Mayo Knee Positioner®

*Part of the De Mayo Positioner® Family*

## Instructions For Use

De Mayo Knee Positioner®  
De Mayo D2 Knee Positioner®  
De Mayo Single Lever Clamp  
De Mayo Push Button™ Clamp  
De Mayo Push Button™ Clamp-Pin Locking




*imp*® ...THE OPERATIVE WORD  
IN PATIENT POSITIONING




# *imp* De Mayo Knee Positioner® Instructions For Use

1. Replace OR Table Pads with green *imp* pads.


*Note: It is not necessary to remove the X-Ray cassette plates.*

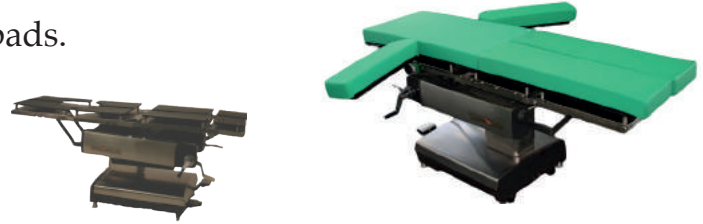


2. Position patient with gluteal fold at separation of pads.

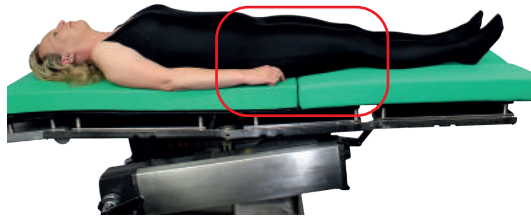

3. During prep of surgical leg, remove single OR Table Pad.

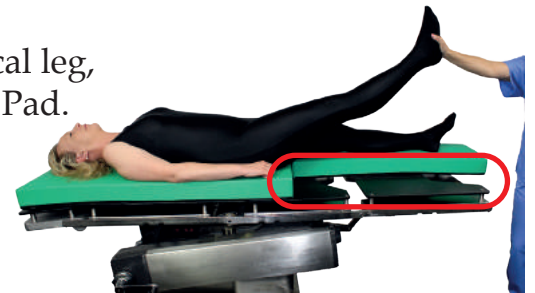

4. Create a 3" - 5" fold in the final drape under the buttocks.

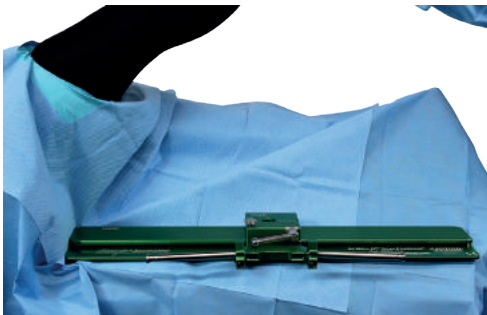

5. Place De Mayo Knee Positioner® in the well to achieve maximum flexion.

6. Check flexion for final approval of freedom of the drape to allow full range of motion when the Positioner is locked on drapes.







# imp® De Mayo Knee Positioner® Instructions For Use

7. Position clamp directly under gliding pins and lock to the OR Table Side Rail utilizing either

De Mayo Single Lever Clamp ~ or ~ De Mayo Push Button™ Clamp ~ or ~ De Mayo Push Button™ Clamp-Pin Locking

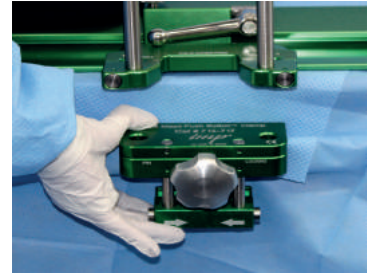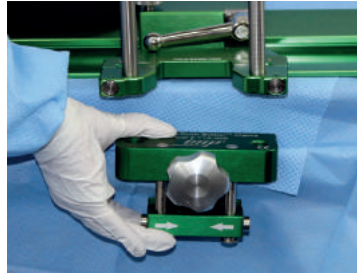

PLEASE REFER TO APPLICABLE CLAMP INSTRUCTIONS FOR USE

8. Protect your patient from potential Pressure Ulcers. Insert the *imp* Patient Protective Pad into the sterile boot.




STERILE EO



9. Wrap cohesive bandage around the foot starting with a minimum of six (6) foot wraps, tear, and finish the wrap above the bracket around the calf.



Avoid wrapping cohesive bandage over the bracket on the back of the boot.

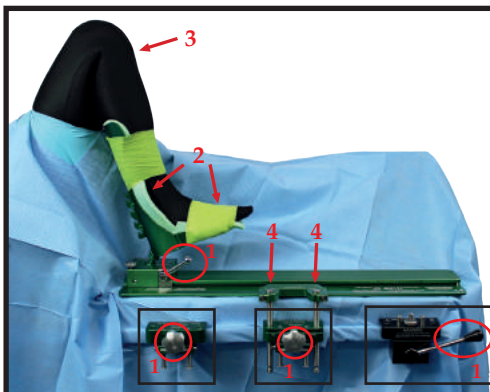

## Set Up Test Procedure



TIGHTEN CLOCKWISE TO LOCK KNOBS AND LEVERS:

- 1 ~ Check all knobs and levers are locked
- 2 ~ Patient is fully protected by imp® foam and cohesive wrap
- 3 ~ Patient's leg can reach full flexion
- 4 ~ Check that both sliding pins are fully seated



imp® products are protected by patent and patent pending rights ~ go to [impmedical.com/patents](http://impmedical.com/patents)  
© August 2019 Innovative Medical Products, Inc. ALL RIGHTS RESERVED

*imp*® Innovative  
Medical Products, Inc.

87 Spring Lane, PO Box 8028

Plainville, CT 06062

PH: (860) 793-0391 FAX: (866) 459-1805


Toll Free PH: 1-800-467-4944

[www.impmedical.com](http://www.impmedical.com)

# *imp* De Mayo D<sup>2</sup> Knee Positioner® Instructions For Use

1. Replace OR Table Pads with green *imp* pads.

*Note: It is not necessary to remove the X-Ray cassette plates.*

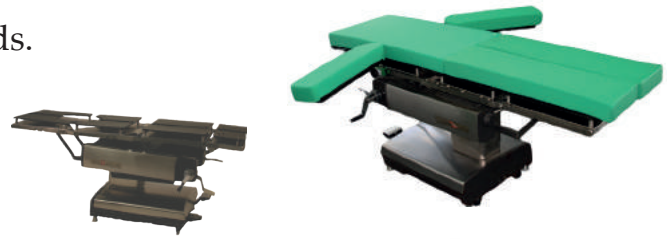

2. Position patient with gluteal fold at separation of pads.




3. During prep of surgical leg, remove single OR Table Pad.



4. Create a 3" - 5" fold in the final drape under the buttocks.



5. Place De Mayo D<sup>2</sup> Knee Positioner® in the well to achieve maximum flexion.

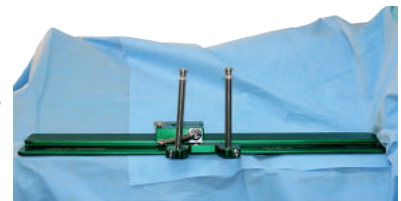

6. Check flexion for final approval of freedom of the drape to allow full range of motion when the Positioner is locked on drapes.






# imp® De Mayo D<sup>2</sup> Knee Positioner® IFU

7. Position clamp directly under gliding pins and lock to the OR Table Side Rail utilizing either the

**Single Lever Clamp ~ or ~ De Mayo Push Button™ Clamp**




PLEASE  
REFER TO  
APPLICABLE  
CLAMP  
IFU

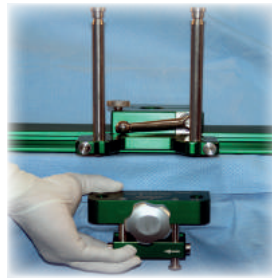

**“Spring Loaded”  
Stainless Steel Handle**  
Additional Resource  
available: refer to IFU



8. Protect your patient from potential Pressure Ulcers. Insert the *imp* Patient Protective Pad into the sterile boot.

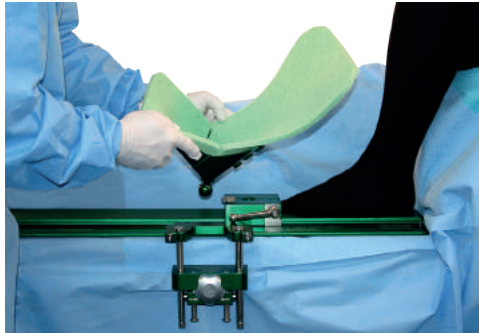

**WARNING**  
Liability Voided if *imp* Patient  
Protective Pads are not used.

9. Wrap cohesive bandage around the foot starting with a minimum of six (6) foot wraps, tear, and finish the wrap above the bracket around the calf.




Avoid wrapping cohesive bandage over the bracket on the back of the boot.



## Set Up Test Procedure

TIGHTEN CLOCKWISE TO LOCK KNOBS AND LEVERS:

- 1 ~ Check all knobs and levers are locked
- 2 ~ Patient is fully protected by *imp*® foam and cohesive wrap
- 3 ~ Patient's leg can reach full flexion
- 4 ~ Check that both sliding pins are fully seated

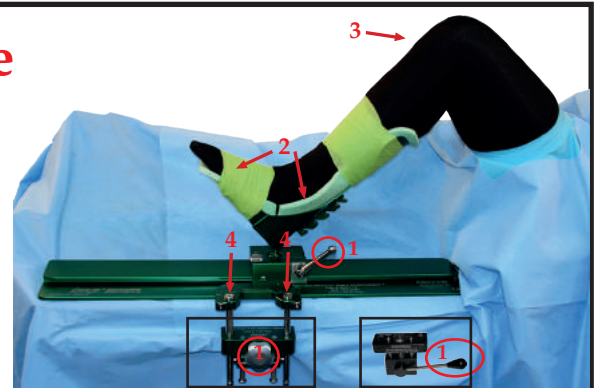

*imp*® **Innovative**  
Medical Products, Inc.

87 Spring Lane, PO Box 8028  
Plainville, CT 06062

PH: (860) 793-0391 FAX: (866) 459-1805


Toll Free PH: 1-800-467-4944

[www.impmedical.com](http://www.impmedical.com)






# *imp* De Mayo Single Lever Clamp Instructions For Use

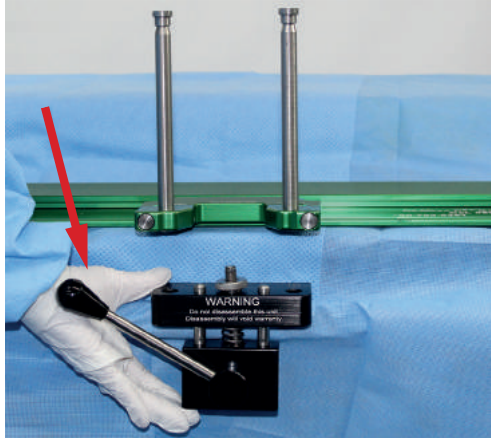

1. Position the clamp holding with the left hand and with the lever in “unlock” position.

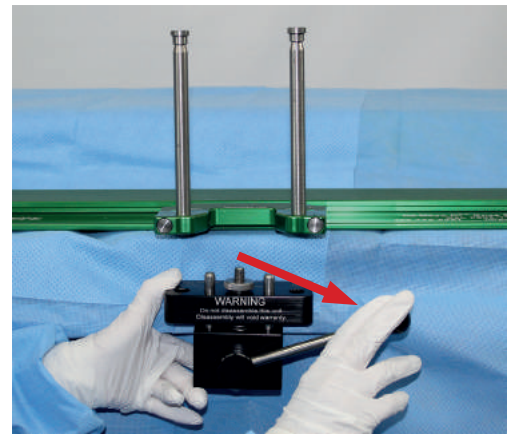


2. Rotate lever to “lock” position.




3. Squeeze jaws tight on side rail, and tighten knurled knob clockwise all the way.

4. While holding clamp, rotate lever to “unlock”.

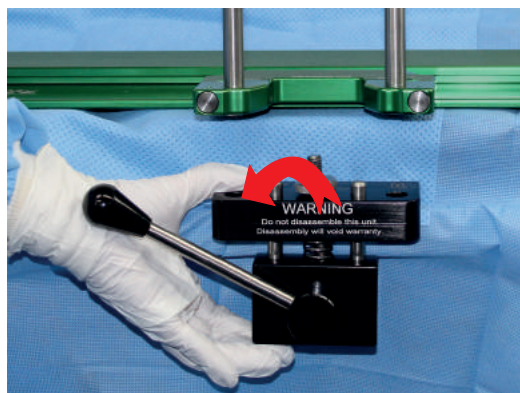





# De Mayo Single Lever Clamp Instructions For Use

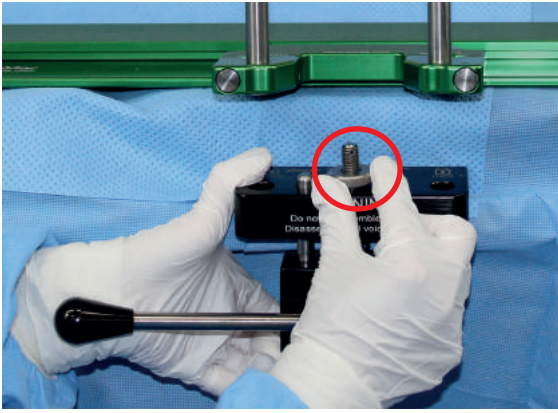


5. Turn knurled knob clockwise 1/4 to 1/2 rotation.

6. Rotate lever to “lock” position requiring only 2 finger pressure to achieve locking. Works like a vice grip and will “snap” into place.

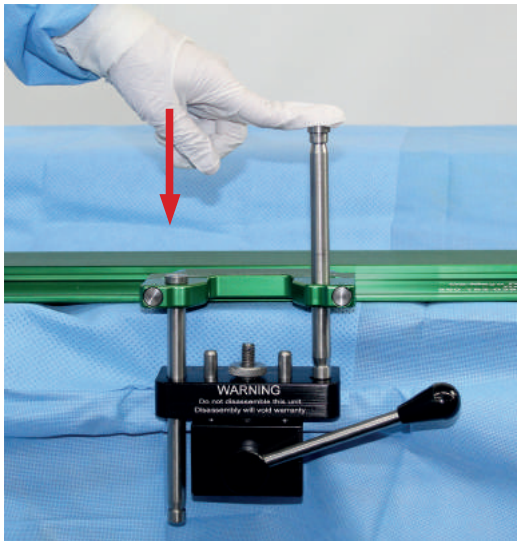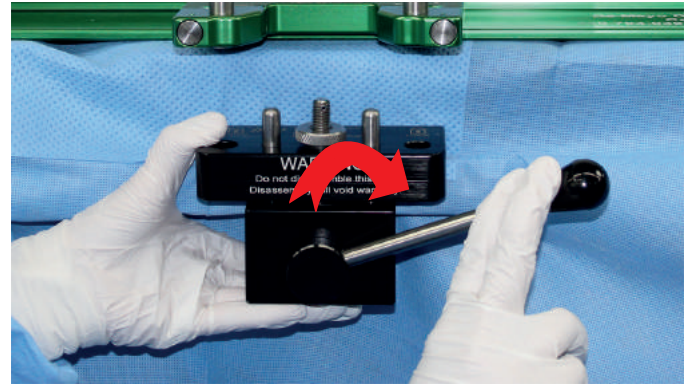

7. Slide and seat pins into holes of the clamp.

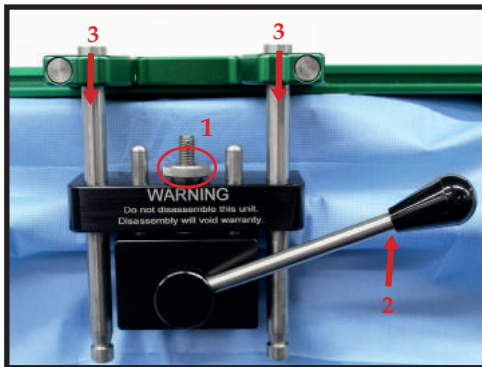

## Set Up Test Procedure

- 1 ~ Ensure clamp knob is tightened
- 2 ~ Lever in the “locked” position to the right side of the clamp
- 3 ~ Check that both sliding pins are fully seated



imp® products are protected by patent and patent pending rights ~ go to [impmedical.com/patents](http://impmedical.com/patents)  
© July 2019 Innovative Medical Products, Inc.  
ALL RIGHTS RESERVED

*imp*® **Innovative**  
Medical Products, Inc.

87 Spring Lane, PO Box 8028

Plainville, CT 06062

PH: (860) 793-0391 FAX: (866) 459-1805

Toll Free PH: 1-800-467-4944



[www.impmedical.com](http://www.impmedical.com)

# imp De Mayo Push Button™ Clamp



## Instructions For Use

De Mayo Push Button™ Clamp


Cat.# 713-302

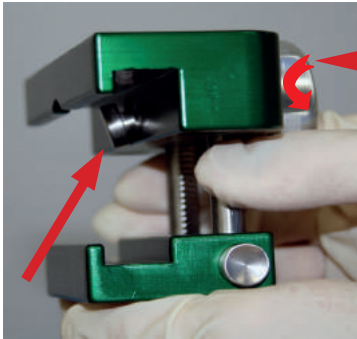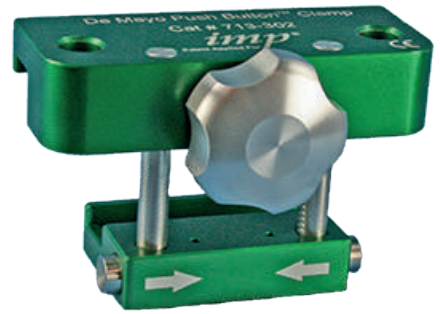


1. Loosen the Clamp Knob all the way by turning it counter clockwise. The leading "cap" should be flush to the upper jaw.

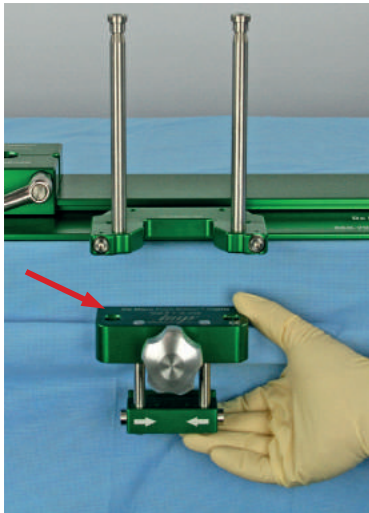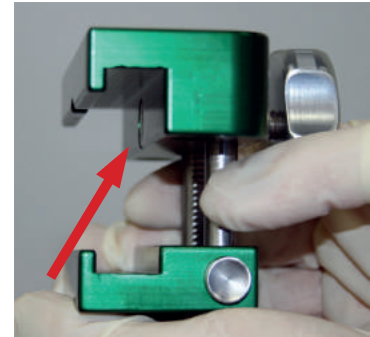

2. Place the Clamp over the drapes with the Clamp holes under sliding pins and "squeeze" the jaws tight on the rail.


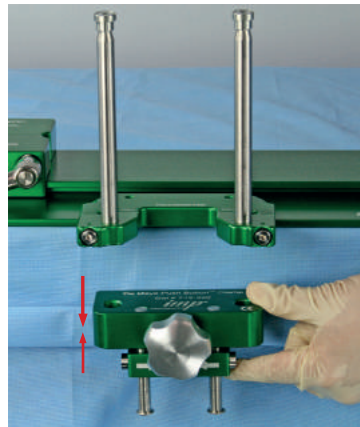


3. Tighten the Clamp Knob by turning it clockwise.

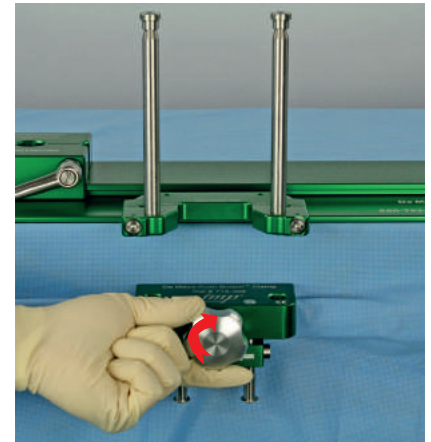

~ locate the Clamp holes under the sliding pins.


4. Seat the sliding pins in the Clamp holes.

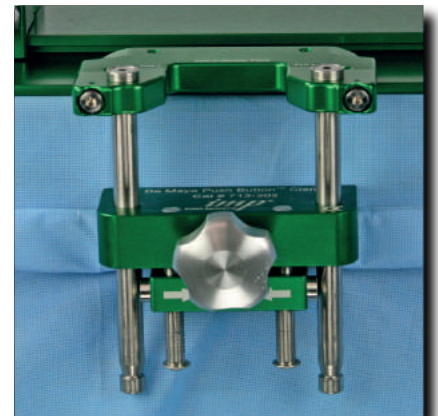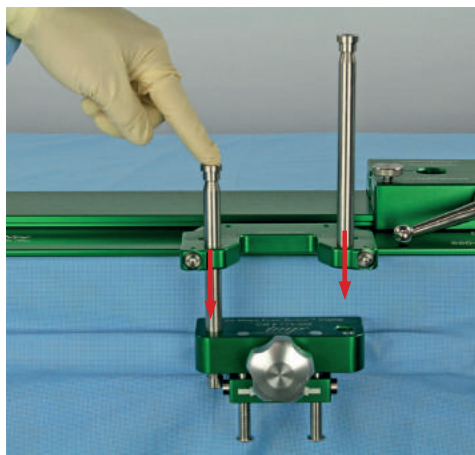



# *imp* De Mayo Push Button™ Clamp


## Instructions For Use

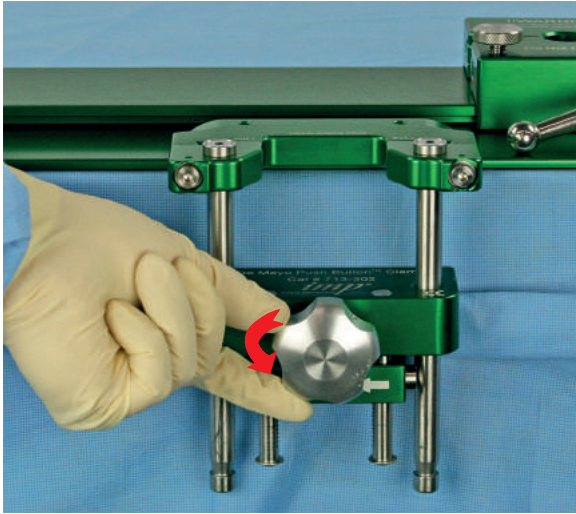


### 1. To REMOVE:

Turn the Clamp Knob counter clockwise until it stops.

### 2. Pull the sliding pins out of the Clamp.

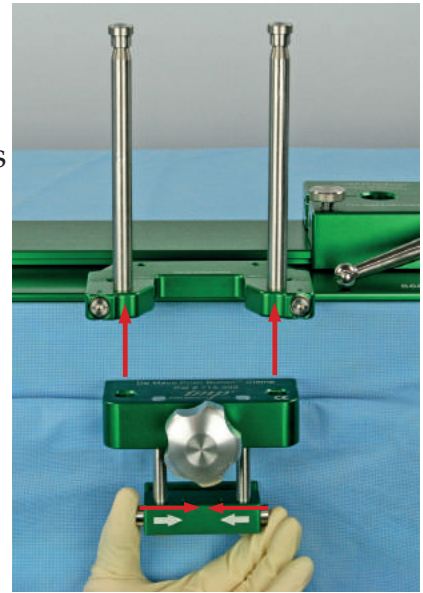

### 3. Lower the bottom jaw by squeezing the release buttons. Remove Clamp.

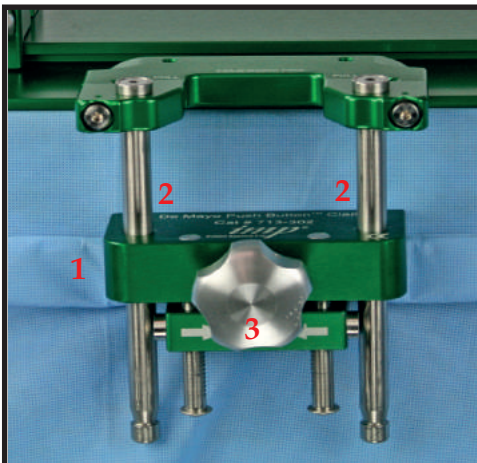

## Set Up Test Procedure



TIGHTEN CLOCKWISE TO LOCK THE KNOBS:

- 1 ~ Check that Clamp is locked on the side rail
- 2 ~ Check that both sliding pins are fully seated
- 3 ~ Check that Clamp Knob is secure



*imp*® products are protected by patent and patent pending rights ~ go to [impmedical.com/patents](http://impmedical.com/patents)  
© December 2017 Innovative Medical Products, Inc. ALL RIGHTS RESERVED

*imp*® **Innovative**  
Medical Products, Inc.

87 Spring Lane, PO Box 8028  
Plainville, CT 06062

PH: (860) 793-0391 FAX: (866) 459-1805


Toll Free PH: 1-800-467-4944

[www.impmedical.com](http://www.impmedical.com)




# *imp* De Mayo Push Button™ Clamp - Pin Locking Instructions For Use

Cat. # 713-717

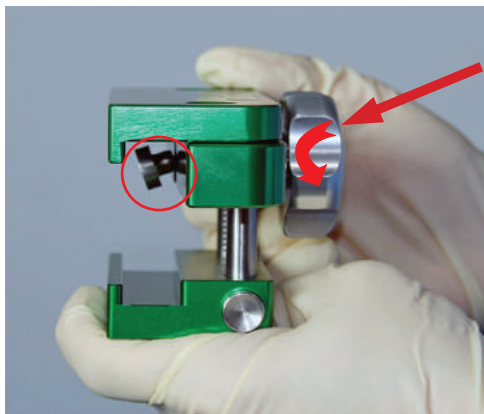


1. Loosen the Clamp Knob all the way by turning it counter clockwise.


○ The leading “cap” should be flush to the upper jaw.

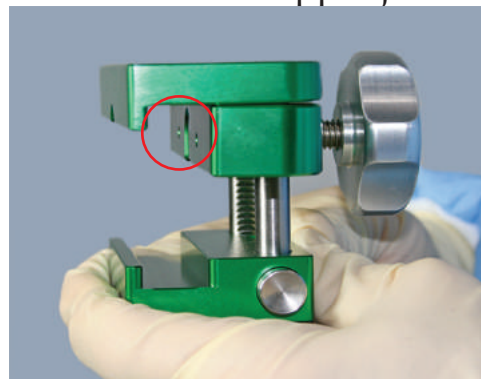


2. Place the Clamp over the drapes with the Clamp holes under the sliding pins

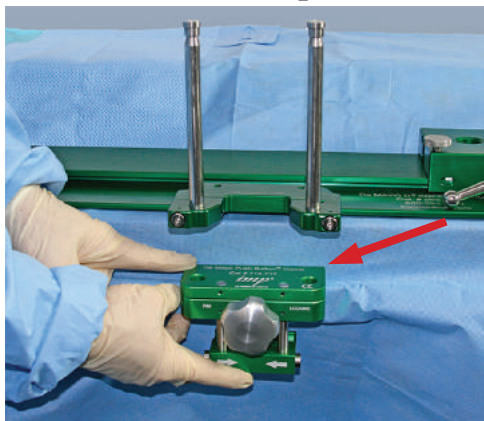


and “squeeze” the jaws tight on the rail.

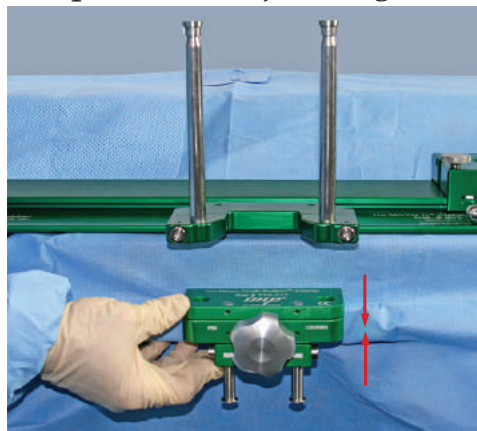

3. Seat the sliding pins into the Clamp holes.

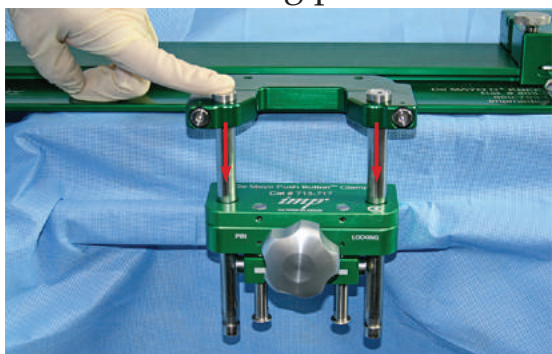


4. Tighten the Clamp Knob by turning it clockwise.

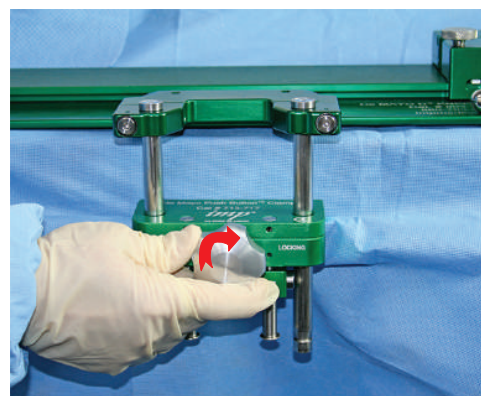





# *imp*® De Mayo Push Button™ Clamp - Pin Locking Instructions For Use

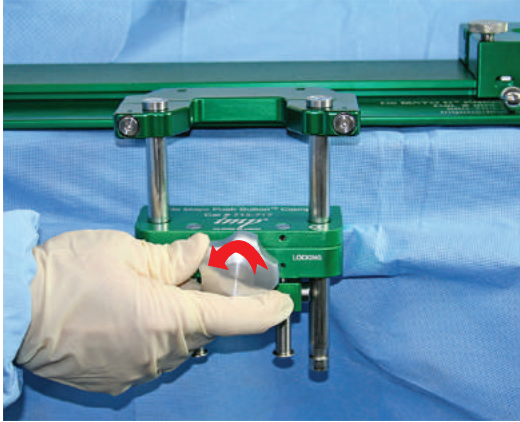

1. *To Remove* - turn the Clamp Knob counter clockwise until it stops.

2. Pull the sliding pins out of the Clamp and lower the bottom jaw by squeezing the release buttons.

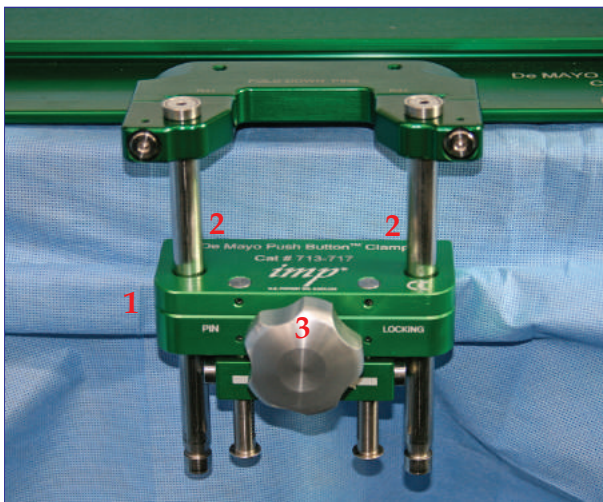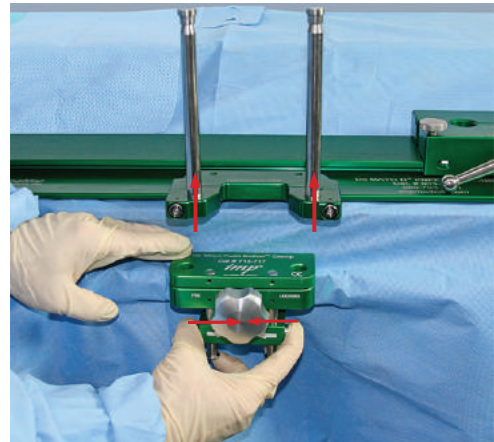

## Set Up Test Procedure

TIGHTEN CLOCKWISE TO LOCK THE KNOB

- 1 ~ Check that Clamp is locked on side rail
- 2 ~ Check that both sliding pins are fully seated
- 3 ~ Check that Clamp Knob is secure





# Value Beyond Patient Protection

~ Certified in Writing ~

[www.impmedical.com/vac](http://www.impmedical.com/vac)

*imp*<sup>®</sup> *Innovative*

*Medical Products, Inc.*

*87 Spring Lane, PO Box 8028*

*Plainville, CT 06062*

*PH: (860) 793-0391 FAX: (866) 459-1805*

*Toll Free PH: 1-800-467-4944*

*www.impmedical.com*

imp<sup>®</sup> products are protected by patent and patent pending rights ~ go to [impmedical.com/patents](http://impmedical.com/patents)

© August 2019 All Rights Reserved